


Work instruction Number 251	
Page 1 of 2	Purpose: Fill and Clean
Safety: <i>Bio Hazard area</i>	
Instrument: Instrument	Revision: 002, 2024

Applicable for Software version 6.12 and up (see WI-197 for procedure for older software versions).


Starrsed Cleaning Agent preparation Instrument Compact: Fill and clean:
This cycle takes about 90 minutes.

1. Fill the clean adapter EHST110907 with hot de-ionized water. (+/- 100 ml, 80 °C)
2. Add 10 ml Starrsed Cleaning Agent (QRR 010905) to the hot water in the adapter.
3. Place the cap on the adapter and mix well.
4. Put the adapter with cleaning solution on the lower tube holder.
5. Select **Maintenance** tab, **Prime/Clean**, button **Fill and clean**.

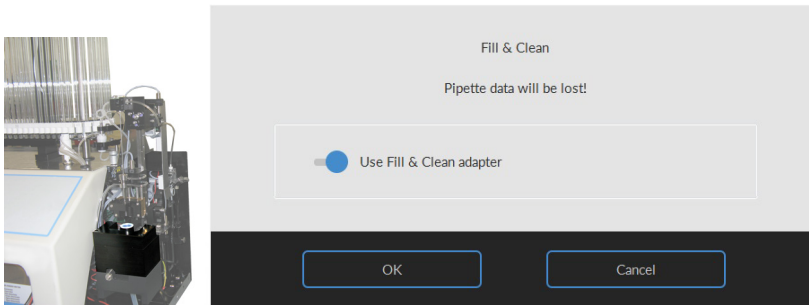

Start Fill and clean procedure:

1. Select button **OK**.
2. The needle goes down and the process is started.
3. When all the pipettes are filled, the needle goes back to the home position.
4. In software version 6.12.1 and up: After the Fill & clean process is finished, a notification is given to stop the procedure.

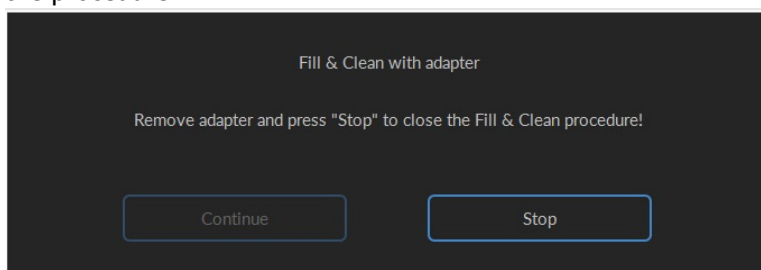

5. Remove the adapter from the needle assembly and press "Stop".

The instrument is ready for normal operation.