

Work instruction Number 406

Page 1 of 1

Purpose: Weekly maintenance

Safety: *Bio Hazard area*

Instrument: Starrsed ST


Revision: 003, March 2019

NOTE: Chip card type **S** is required !

Prepare disinfectant: (if not already prepared).

This disinfectant is for cleaning of all external parts that are exposed to blood.

1. Wipe the lower tube holder of the needle unit.
2. Wipe the stainless steel drip plates below the pipette carousel and needle unit.

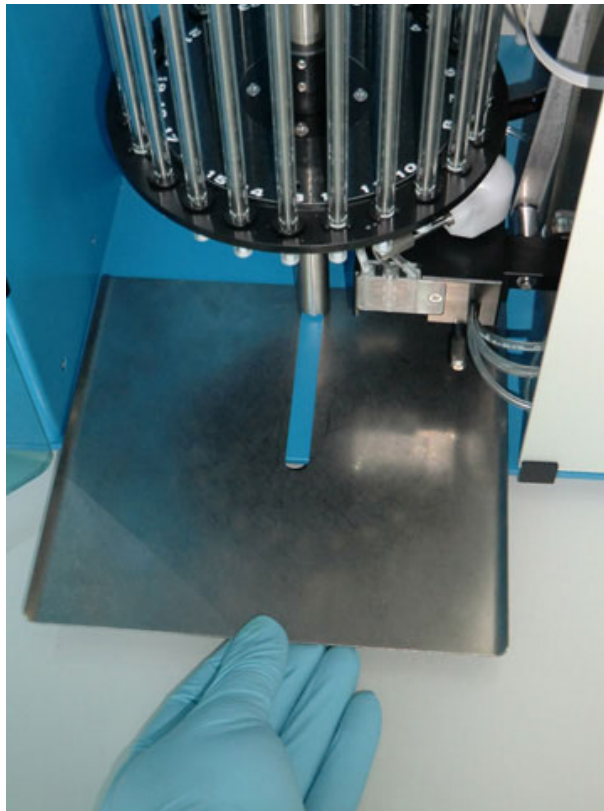

3. In the Main menu use the function [END_OF_DAY WASH] .
4. Check the system for leakage.
5. Inspect sample needle condition:
Menu SERVICE - MOTOR CONTROL - NEEDLE M31 Press [**ARROW DOWN**] twice to expose the needle tips. If the tips are worn, replace the needle.
6. Check sensors:
Menu SERVICE - NEXT - SENSOR STATUS
Blood sensor: value 110-160
Measure head: value 40-60